

# Adhesive for dental prostheses

With Hyaluronic Acid and Chamomile Oil

MEDICAL DEVICE

CLASS I

OBSERVATIONAL  
CLINICAL STUDY

Other areas



PACKAGING  
50 ml

## HOW TO USE

Clean the dental prosthesis with running water and dry it carefully. Apply small quantities of product, keeping away from the edges. Reposition the dentures in your mouth and hold them in place for a few moments, waiting a few minutes before eating or drinking. Remove any residual adhesive paste from the dentures or mouth with hot water and, if necessary, with a toothbrush or cotton swab soaked in oil.

## PRECAUTIONS FOR USE

the duration of the adhesive power may vary depending on personal physical characteristics. Apply no more than twice a day. In order not to clog the dispenser, keep the cap and spout dry and clean. Do not swallow.

**IT PREVENTS THE INFILTRATION OF FOOD RESIDUES**  
**ENSURES THE LOCKING OF THE MOBILE BRIDGE**  
**IT SOOTHES THE MUCOSA BY PREVENTING IRRITATION OF THE GUMS**  
**WITHOUT PRESERVATIVES**

## INGREDIENTS

CALCIUM / SODIUM PVM / MA COPOLYMER, PETROLATUM, CELLULOSE GUM, PARAFFINUM LIQUIDUM, PVM / MA COPOLYMER, CHAMOMILLA RECUTITA FLOWER EXTRACT, SODIUM HYALURONATE.