

Fitoeye Sterile Monouse Wipes

It is indicated for daily periocular hygiene and in case of irritation from contact lenses; also as an aid in conjunction with the drug therapy in case of:

Blepharitis, Conjunctivitis, eye allergies, stye (infection of eyelid), Eye traumas and post operation (being Sterile)

MEDICAL DEVICE

CLASS I STERILE

Ophthalmology



PACKAGING
20 monouse
sterile wipes

Retail public price
in Italy: € 10.50

Tea Tree Oil Tit. 30% Terpen 4-OI

Selective antibacterial activity on the main bacteria responsible for eyelid and eyelash infections (staphylococcus aureus, s. epidermidis and pseudomonas aeruginosa); Anti-inflammatory activity; Acaricidal activity VS DEMODEX main cause of blepharitis (inflammation of the eyelids)

Echinacea Angustifolia emollient, anti-inflammatory, re-epithelizing activity.

**STERILE
PARFUM FREE
PRESERVATIVE FREE**

HOW TO USE

Open the sachet starting from the pre-cut cutting invitation with clean hands, remove the wipe and massage it gently on the eyelids and eyelashes keeping the eye closed. Use a wipe for each eye, after use discard the wipe. Do not rinse after application. The closed sachet can be heated in warm water (about 40° C) to use the wipe as a hot compress and remove particularly persistent residues, or cooled in the refrigerator, to apply the wipe as cold compress in case of inflammation.

INGREDIENTS

AQUA, LAURYL GLUCOSIDE, LAURETH-9, CARBOMER, PROPYLENE GLYCOL, GLYCOLIC EXTRACT OF MIMOSA, GLYCOLIC EXTRACT OF ECHINACEA, CETEARETH-20, CETYL ESTER, PHENYLETHYL ALCOHOL, CAPRILYL GLYCOL, EDTA DISODIUM, TRIETHANOLAMINE.